

REGAL GO™

ALL-IN-ONE ORAL CARE BRUSH

Regal GO™ is the all-in-one oral care brush that helps break up and rinse away food debris in even the most hard-to-reach areas.

INSTRUCTIONS



Step 1: Twist the Regal GO pen to extrude gel onto the brush tip. It may take a few extra turns on a brand new pen!



Step 2: Apply gel directly to your teeth, under prosthetics, or in and around implants or orthodontic brackets.



Step 3: Use the brush tip to gently scrub and dislodge food debris. Repeat as needed.

Step 4: Vigorously rinse and swish with water to maximize the foaming and cleaning action.

Replace the cap, and keep it anywhere you want to be able to quickly freshen your smile!

INGREDIENTS

Glycerin, Aqua (water), Sorbitol, PVP, Xylitol, Cocamidopropyl Betaine, Ammonium Acryloyldimethyltaurate/VP Copolymer, Xanthan Gum, Potassium Phosphate Dibasic, Disodium EDTA, Hydroxyethyl Cellulose, Sodium Benzoate, FD&C Blue, Sodium Lauryl Sulfate, Sodium Saccharin, L-menthol, Aroma (flavor)

WARNING

Keep out of reach of children under 6 years of age. Do not ingest. If more than is used for brushing is accidentally swallowed, get medical help or call a Poison Control Center right away.



**A SUBSCRIPTION IS A
PRESCRIPTION FOR SUCCESS!**

Save big and never run out when you
subscribe and save at regal goclean.com

*Scan the QR code
below to learn more!*

Manufactured by:

**ZD ZEST DENTAL
SOLUTIONS**

Zest Dental Solutions
2875 Loker Ave E
Carlsbad, CA 92010
www.zestdent.com
888.734.2546



09430 REV A 2023-03